






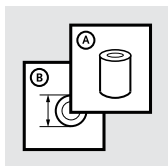
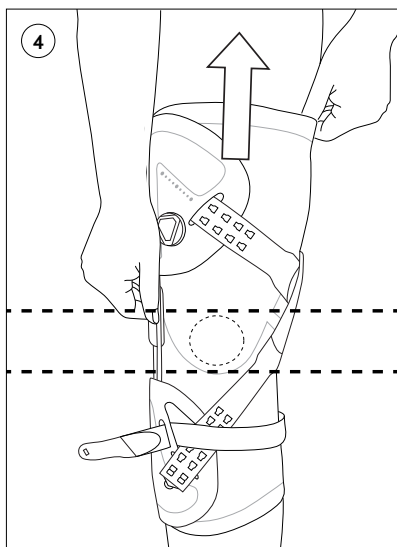
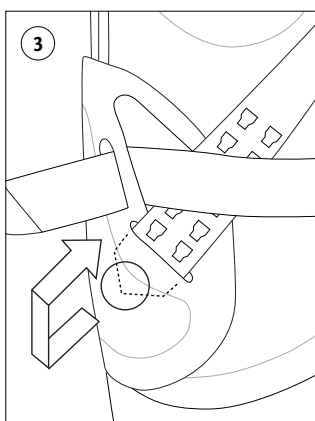
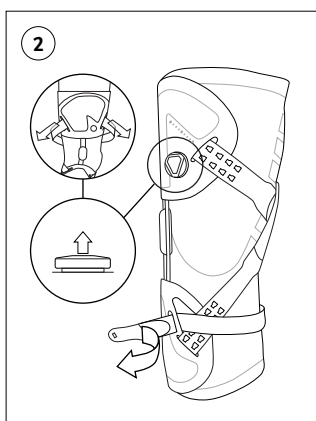
Instructions for Use

UNLOADER[®] ONE LITE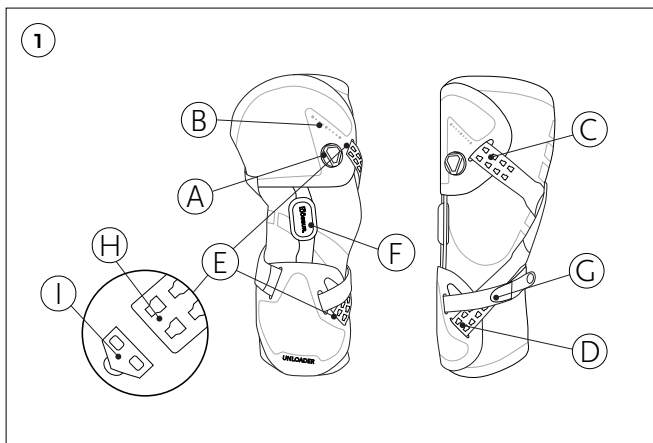

		3
EN	Instructions for Use	6
DE	Gebrauchsanweisung	8
FR	Notice d'utilisation	11
ES	Instrucciones para el uso	14
IT	Istruzioni per l'uso	17
NO	Bruksanvisning	20
DA	Brugsanvisning	23
SV	Bruksanvisning	26
FI	Käyttöohjeet	29
NL	Gebruiksaanwijzing	32
JA	取扱説明書	35
ZH	中文说明书	38
KO	사용 설명서	40

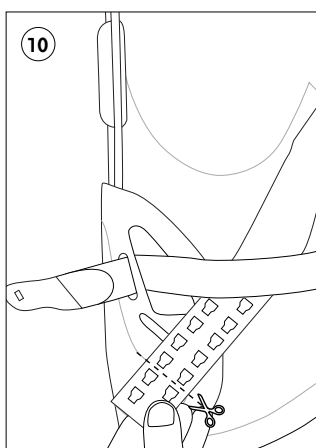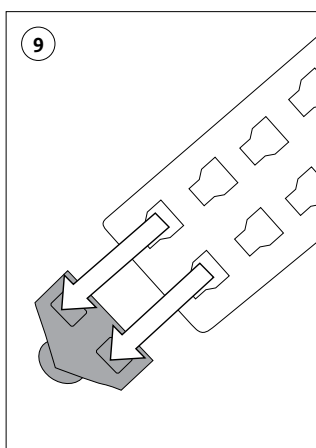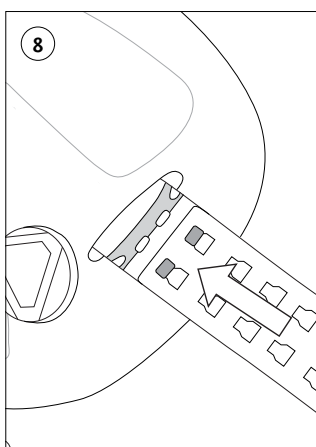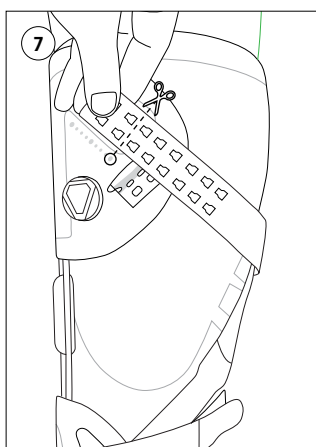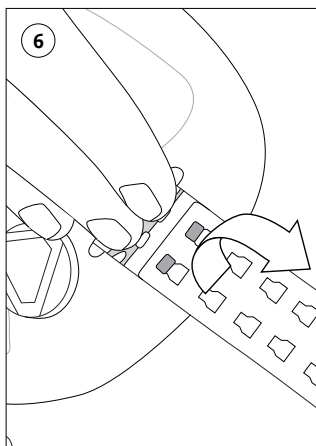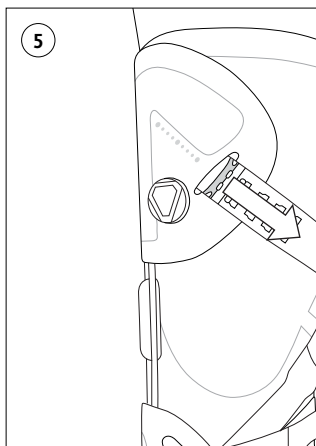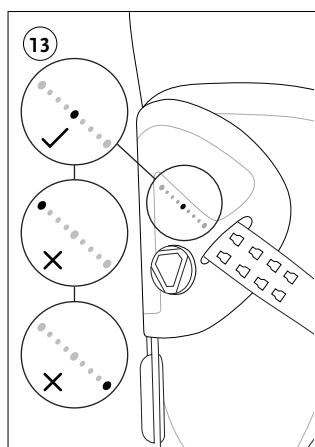
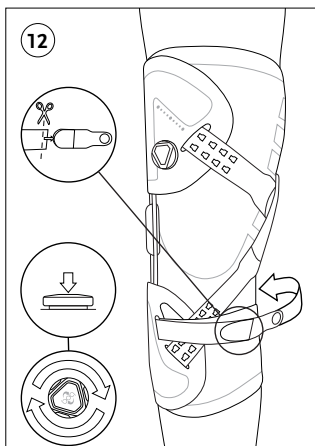
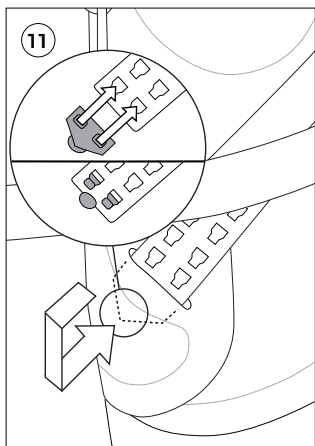
FIGURE 1

- A. SmartDosing® Dial
- B. SmartDosing® Scale with Indicators
- C. Upper Dynamic Force System (DFS) Strap
- D. Lower Dynamic Force System (DFS) Strap
- E. Sleeve Slots
- F. Polycentric Hinge
- G. Calf Strap
- H. Button Hook Holes
- I. Hook Tab

INDICATIONS FOR USE

- Mild to moderate unicompartmental knee osteoarthritis
- Degenerative meniscal tears
- Other knee conditions that may benefit from unicompartmental unloading

CAUTION

For users with peripheral vascular disease, neuropathy, and sensitive skin, close practitioner supervision is recommended. If redness appears, decrease usage of brace. Increase usage slowly as skin adapts.

UNLOADER ONE® LITE is a field adaptable knee brace, and should be modified to fit individual patient contour by an orthotist or qualified practitioner. A proper fit is key to achieving effective pain relief from the brace.

INITIAL APPLICATION AND ADJUSTMENT

Preparation and Donning (Figures 2–4)

- Before donning brace, pull out SmartDosing® dial to release.
- Undo calf strap and loosen both DFS straps by pulling both at the same time.
- Detach lower DFS strap from shell by sliding hook tab downward, then lifting it up from shell hole and out from sleeve (strap stays attached to hook tab during this movement).
- Ask user to don brace while seated. Then in standing position, align brace such that patella top is level with mid-hinge.

Upper DFS Strap Adjustment (Figures 5–8)

- Pull upper strap slightly further, until yellow indicator component is accessible.
- Hold yellow component in place while detaching upper strap from button hooks.
- Estimate ideal length of strap by placing it next to the SmartDosing scale and identifying cut-line aligned with the first indicator.
- Trim strap, making sure to leave at least one extra set of holes above sleeve slot.
- Round corners of strap by trimming, and reattach to button hooks.
NB: If unable to trim straps immediately, attach to appropriate set of holes and thread into sleeve slot.

Lower DFS Strap Adjustment (Figures 9–11)

- Detach strap from hook tab.
- Place strap on leg next to sleeve slot.

- Ideal trimline is at least one set of holes beneath sleeve slot.
- Reattach DFS strap to hook tab, making sure it is fully fastened.
- Insert hook tab into sleeve slot, hook into shell and pull lightly to secure in place. **NB:** If unable to trim straps immediately, attach to appropriate set of holes and thread into sleeve slot.

Assessment of Strap Tension (Figure 12–13)

- Fasten calf strap above widest part of calf.
- Push down SmartDosing® dial, and ask user to slightly bend knee.
- Turn dial clockwise until yellow display is at mid-indicator.
- Ask user to sit down, stand up, and take a few steps.
- User should feel strap tension without discomfort at mid-indicator setting.
- If indicator is at top setting without proper tension, straps will need to be trimmed further.
- Indicator at bottom setting means strap has likely been cut too short.

NB: Straps will elongate slightly during use and may need to be adjusted further after a few weeks of use. DFS straps with alligator clips are available as an accessory, see QRC 1241_003 for application.

Final Check

Verify positioning of brace when adjustment procedure has been completed.

DoFFing

Pull out SmartDosing® dial to release, loosen both DFS Straps simultaneously, then Calf Strap.

Remove brace by gently pulling on hinge and opposite lower edge of brace.

RANGE OF MOTION ADJUSTMENT (ROM)

The device comes with a 0° extension stop installed.

Flexion/Extension Stop accessory kit is available under item number B-301040047.

Note: If extension stop is modified, DFS strap tension needs to be adjusted accordingly.

WASHING INSTRUCTIONS

Hand wash using mild soap and rinse thoroughly. For machine wash, use a wash bag, and program for delicate wash. Air-dry only. Do not tumble dry or use bleach.

MAINTENANCE

- For safety reasons, hinge should only be disassembled by a certified practitioner.
- Clean dirt from hinge using tap water and/or compressed air.
- Avoid contact with salt water or chlorinated water. If such contact occurs, rinse well with tap water and air dry.
- Regular cleaning will ensure hinge, sleeve, straps, and pads are in optimum working order.

PRODUCT MATERIAL LIST

Thermoplastic elastomer, polypropylene, nylon, polyurethane, polyester, aluminum, stainless steel, cyanoacrylate.

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